

recomLine TORCH Screening IgG, IgM sample strips

			Cut-off Contr.												Interpretation	
			Conjugate Contr.		<i>T. gondii</i>		Rubella		CMV		HSV					
			IgG	IgM	lysate	p30	lysate	lysate	lysate	gB2	lysate	lysate	lysate	lysate		
			IgG	IgM	lysate/ROP1c	/	lysate	lysate	lysate	/	lysate	lysate	lysate	-		
1	Rubella seronegative	IgG														Seronegative; no immunity for Rubella
2		IgM														
3	Rubella acute infection	IgG														No protective immunity for Rubella: Suspicious of primary infection
4		IgM														
5	Rubella past infection	IgG														Protective immunity for Rubella: Suspicious of past infection
6		IgM														
7	Toxoplasma seronegative	IgG														Seronegative; no immunity for Toxoplasma
8		IgM														
9	Toxoplasma acute infection	IgG														Suspicious of recent Toxoplasma infection
10		IgM														
11	Toxoplasma past infection	IgG														Suspicious of past Toxoplasma infection (p30 positive: > 3 month p.i.)
12		IgM														
13	CMV seronegative	IgG														Seronegative; no immunity for CMV
14		IgM														
15	CMV acute infection	IgG														Suspicious of recent CMV infection
16		IgM														
17	CMV past infection	IgG														Suspicious of past CMV infection (gB2 positive: > 6-8 weeks p.i.)
18		IgM														
19	HSV Type 2 infection	IgG														Suspicious of HSV type 2 infection
20		IgM														